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GB Important instructions

This leaflet contains vital information which must be observed!

Definition of symbols on the packaging

Caution – refer to instructions

Batch number

Use by
(year, month)

Introduction

The NUK Therapeutic Trainer is a therapeutic instrument which makes an important contribution to overcoming suction weakness and normalising defective oral motor activity (e.g. oral breathing as well as suction, chewing, swallowing, speech and mimicking gesture disorders). In addition, using the NUK Therapeutic Trainer can have a positive effect on the regulation or harmonisation of the intraoral and perioral musculature. Persistent habits such as grinding teeth, clenching the teeth and tongue associated with pain and cracking of the jaw joints can also be effectively treated.

The NUK Therapeutic Trainer can also be used for speech, voice and breathing therapy applications.

The product is available in 2 sizes

- Size **S** for schoolchildren
- Size **L** for adults

For use in myofunctional therapy according to PADOVAN, the mouthplate can be supplied with holes through which thin catheters can be inserted (see "Holes in the mouthplate") to facilitate the introduction of small amounts of water for certain suck-swallow exercises.

Important information

The NUK Therapeutic Trainer for use in speech therapy and physiotherapy is a Class 1 medical device (Directive 93/42/EEC).

- The Therapeutic Trainer is manufactured from natural rubber latex which may cause allergies.
- The Therapeutic Trainer must be thrown away if it is damaged or there is visible aging (sticky, cracking).

The following conditions apply to usage

- The nature and duration of usage is determined by the responsible specialist.
- It must only be used under expert supervision following prior instruction and training.
- Clean and disinfect the Therapeutic Trainer thoroughly before each use. Squeeze out any excess liquid. When sterilising the Therapeutic Trainer, ensure that there is always sufficient liquid in the container so that the Therapeutic Trainer is not damaged (approx. 5 minutes sterilisation time is sufficient). The water must not be allowed to boil away.
- The Therapeutic Trainer must not be cleaned in the dishwasher as the detergent may cause material damage.
- The Therapeutic Trainer must not be placed in the microwave – there is a risk of explosion/material damage.
- The Therapeutic Trainer must not be used if there is a possibility of anaphylactic shock.

• The Therapeutic Trainer is not suitable for administering medication.

• The Therapeutic Trainer must not be used in the patient's mouth at the same time as other medical equipment (with the exception of the catheters listed above), medicines or care substances. It is particularly important to prevent contamination with medication or care substances, as interactions between the materials cannot be excluded. As a general rule, individual cases should be discussed with the responsible specialist.

- The Therapeutic Trainer must not be allowed to come into contact with greases or oils, as this may restrict its usability.
- If stored correctly, the Therapeutic Trainer can be used up to the date printed on the individual wrapping.

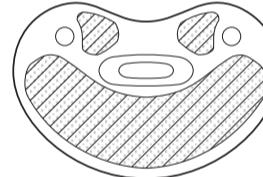
• The Therapeutic Trainer should be stored hygienically and protected from direct sunlight in a cool, dry and dark place. Avoid storage locations where the temperature and air humidity is significantly above room level (in relation to moderate European climates). A continuous supply of heat will substantially accelerate the natural ageing process. The UV radiation component of sunlight, as well as artificial UV rays, will likewise accelerate the natural ageing process. When not required, the Therapeutic Trainer should be stored in a cupboard etc., where possible.

- Before each use, it is vital that you check the Therapeutic Trainer for damage. Check its strength by pulling firmly on the sucking part.
- If damaged, the Therapeutic Trainer should be discarded and replaced. We recommend that the Therapeutic Trainer should be changed regularly.
- Keep the packaging and this leaflet.
- Do not leave the Therapeutic Trainer in disinfectant (sterilizing solution) for longer than recommended as this may weaken the Therapeutic Trainer.

Holes in the mouthplate

The NUK Therapeutic Trainer can be used in therapy in combination with a thin catheter for suck-swallow exercises.

The holes required for this can be made with conventional HSS twist drills for metal processing which are available from specialist tool shops. We recommend drilling a maximum of 2 holes, each with a maximum diameter of 3 mm, in the mouthplate. Only drill in the areas of the mouthplate shaded in the diagram. Maintain a minimum distance of ≥ 5 mm from the edge of the plate.

If more than 2 holes are made or the holes have a diameter in excess of 3 mm, we cannot provide a manufacturer's warranty. The holes must be trimmed to ensure that the user does not incur any injury.

Thank you very much for your attention.

NUK Clinic Advisory Service

Please keep address, art. No. and batch No. in case of complaint.

Distributed in the UK by MAPA Spontex UK Ltd., Berkeley Business Park, Wainwright Road, Worcester WR4 9ZS, Helpline: 0845 300 2467, www.nukbaby.co.uk

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