D

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MAPA GmbH
Industriestraße 21-25
27404 Zeven
Tel.: 08002007615
www.nuk.com

GB

Important information before using for the first time

Read the warnings and the instructions for use carefully.

Retain the packaging and this pack insert.

1. Product description

1.1 Proper use

NUK Training Soothers are oversized soothers intended for schoolchildren and adults. They are designed to satisfy the need to suck not serving for the intake of food and to train the oral motor functions.

2.2 Hygiene

Clean the Training Soother before each use and sterilise it (see "3. Cleaning, sterilising, storing").

3. Reinigen, desinfizieren, lagern

3.1 Reinigen

Do not immerse the Training Soother in medicine or sweet substances and do not suck off the Training Soother. Both can cause tooth decay.

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Properly wash the Training Soother. Do not expose the Training Soother to direct sunlight or heat.

3.3 Lagern

Keep the Training Soother clean, dry and cool away from light. Select storage locations with a temperature and humidity at or below those of the ambient atmosphere.

3.4 Kontakt

Please retain the product information and Article/LOT No. for future use.

Service Centre

Newell Brands UK Limited
Halifax Avenue, Bradley Park
Lichfield, Staffordshire, WS13 8SS
United Kingdom www.nuk.co.uk

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3. Cleaning, sterilising, storing

3.1 Cleaning

Clean the Training Soother in just water. If it is heavily soiled, use a mild detergent.

3.2 Sterilising

Boil in water:

Approx. 5 minutes is sufficient boiling time. Make sure that the water level in the pan is always sufficient during boiling to prevent the Training Soother being damaged. The water must not boil dry. Squeeze any excess liquid out of the part which is sucked.

3.3 Lagern

The Training Soother should only be used under professional supervision following prior instruction and introduction.

3.4 Kontakt

Please retain the product information and Article/LOT No. for future use.

Service Centre

Newell Brands UK Limited
Halifax Avenue, Bradley Park
Lichfield, Staffordshire, WS13 8SS
United Kingdom www.nuk.co.uk

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